PREHEATER 1.0

INSTRUCTIONS FOR USE | INFORMATIONS CONCERNANT L'UTILISATION | GEBRAUCHSINFORMATION

ENGLISH - TABLE OF CONTENTS

1. Technical description	5
1.1 Scope of delivery	5
1.2 Device description	5
1.3 Technical data	6
2. Safety information	6
2.1 General safety information	6
2.2 Device-specific safety information	7
3. Installation	8
3.1 Description of the control panel	8
3.2 Function keys	8
4. Operation	8
4.1 Start-up operation	8
4.2 Temperature setting	9
4.3 Temperature for use	9
4.4 Color mixing	10
5. Troubleshooting	10
6. Maintenance	10
7. Guarantee	11
8. Disposal	13
9. EC declaration of conformity	14

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SYMBOLS USED

The symbols used in the text are defined below.

Symbol for a hazard

- ▶ Immediate danger to life
- ▶ Risk of serious personal injury
- ▶ Risk of minor personal injury

Symbol for a hazard

- ▶ Risk of damage to property
- ▶ Risk of damage to the environment

Symbol for useful advice and information

Symbol for a necessary activity

INTENDED USE

The PreHeater 1.0 is a device for directly heating HyCeram® or Luxon Design materials. These materials can be heated directly in the can or cartridge in a fast and effective manner. This considerably improves the flowability of the materials and makes it easier to mix and apply the materials in corners and edges of jewellery or watch components.

The PreHeater 1.0 is an integral system component of the HyCeram® and Luxon Design systems and is a key element for their successful and practical deployment.

1. TECHNICAL DESCRIPTION

1.1 Scope of delivery

- ▶ PreHeater 1.0
- ▶ Power cable with plug
- ▶ Glass mixing plate

1.2 Device description

1. Main switch
2. Control panel with display and programming keys
3. Heating block
4. Glass mixing plate
5. Heating segments for cans and mixing plates
6. Heating inlets for cartridges 50 g
7. Heating inlets for cartridges 8 g
8. Type plate information

1. TECHNICAL DESCRIPTION

1.3 technical data

Power connection	230 V/50-60 Hz
Heat output	180 W
Temperature range	max. 70 °C
Sound pressure level	≤ 70 dB(A)
Housing material	Stainless steel

2. SAFETY INFORMATION

2.1 General safety information

- ▶ Each device leaves the factory after final testing and a test run. However, if the device is damaged or affected by transportation or other circumstances, it must not be used. Check the device for damage at regular intervals. Damaged parts must be replaced immediately.
- ▶ All electrical and other supply connections must be connected in accordance with the technical data. The values must not be exceeded or fall below specifications. Take VDE regulations into account. Only use approved grounding connections.
- ▶ The regulations regarding any tests that may be required (e.g. in accordance with trade associations or electrical equipment) are the sole responsibility of the purchaser/user of the product.
- ▶ This device is designed exclusively for use in accordance with the technical description and therefore is only to be used for the intended purpose. Any other use is not permitted for safety reasons – the manufacturer shall not be liable in this regard.
- ▶ The device may only be operated by persons trained in how to use it and who are expressly commissioned to operate it.
- ▶ The installation surface must be level, even and suitably stable for use.
- ▶ With the exception of processing materials, no other items may be placed on the device.
- ▶ If the expected work results are not achieved, stop using the device. The owner is responsible for all consequential damage caused by continued use.

2. SAFETY INFORMATION

- ▶ In order to avoid damage from water, fire and explosions, switch off all supplying power sources immediately after finishing work or if work is interrupted and secure against unauthorised activation.
- ▶ Always switch off the main switch and pull out the plug before maintenance, cleaning and repair work.

2.2 Device-specific safety information

- ▶ Place the device on a stable, flat and water-repellent surface.
- ▶ The device can reach temperatures of up to 70 °C. Make sure you maintain sufficient safety clearance when you are near the device.
- ▶ No flammable or explosive substances may be stored in immediate proximity to the device.

Notes :

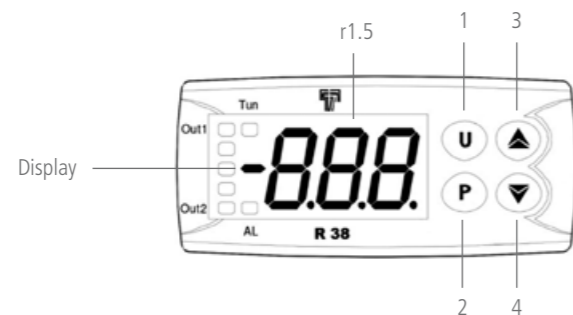
In order to avoid injuries when using the device, ensure compliance with the corresponding protective measures.

3. INSTALLATION

 The device is delivered fully installed and can be operated with a standard 230 V/16 A connection. Note the safety information for installation in section 2.

3.1 Description of the control panel

The device performs a self-test after it is switched on. The following display is shown on the screen for 5 seconds: r1.5. Then the actual temperature (°C) is shown.

Ill. : function field

3.2 Function keys

- 1 Only for programming in the factory
- 2 Program key
- 3 Increase temperature
- 4 Decrease temperature

4. OPERATION

4.1 Start-up operation

-  ▶ Connect the power supply with the main plug on the device. Switch the device on with the main switch (1).
-  ▶ Switch the device off with the main switch (1) when the work is finished.

4.2 Temperature setting

To set the temperature, proceed as follows:

-  ▶ Press key 4. The display flashes alternately between SP1 and the target temperature.
-  ▶ Set the required target temperature with keys 3 (increase) and 4 (decrease).
-  ▶ Press the key P (2) to save the target temperature.

 **Note:** The set value will be saved as the temperature target value until a new one is entered, even if the device is switched off or if there is a power cut.

 **Note:** Without a temperature setting, the device automatically switches to the original heating mode and temperature target value after 15 seconds.

4.3 Processing temperature and correct handling

 The ideal temperature for use or mixing for HyCeram® and Luxon materials is **50 °C - 60 °C**. With a warm-up time of approx. 15 - 20 minutes.

 **Warning:** the highest temperature must not exceed 70 °C.

 **Note:** Pearl and Luminex colors set quickly at elevated temperature. Therefore it is recommended to turn the cartridge once over 180 °.

 **Warning:** the highest temperature must not exceed 70 °C.

4.4 Color mixing

- ▶  Insert the cartridges into the inlets (6/7) of the material block (3) or place the cans on the heating segments (5). To mix the material, squeeze the appropriate HyCeram® or Luxon materials into the glass mixing plates and mix them carefully using a suitable spatula.

 **Note:** mix carefully in circular motion to prevent trapped air.

5. TROUBLESHOOTING

	Device cannot be switched on.	a. Check building fuse. b. Check device fuse.
	Display jumps back.	Set and confirm the target temperature. See 4.2. of instructions for use.
	Material flows too thinly	Reduce target temperature.
	Material doesn't flow enough	Increase target temperature.

6. MAINTENANCE

- ▶  Regularly check the status of the device. Take note of visible damage to the housing or power cable.
- ▶  Clean the housing at regular intervals with a damp cloth or with stainless steel care spray.

7. GUARANTEE

The guarantee period is 2 years from the delivery date. The guarantee applies only to defects in material, manufacturing and processing. The guarantee does not cover natural wear and tear, or damage after the transfer of risk as a result of incorrect or unsuitable use, excessive use, unsuitable operating equipment or the effects of unnecessary chemical, electrochemical or electrical influences. Improper modifications or maintenance or repair work on the part of the operator or third parties shall invalidate guarantee claims. Consumer rights arising from this guarantee are limited to the claim for supplementary performance. This guarantee does not permit further claims, particularly for any type of compensation.

1. Guarantee claims

If you wish to make a guarantee claim, please contact your invicon specialist retailer or us directly at:
 invicon chemical solutions gmbh
 schweizer strasse 96
 6830 Rankweil (austria)
 t : +43 5522-45301-0
 e : office@invicon.at

Guarantee claims must be submitted as follows.

- ▶ Specification of serial number
- ▶ Original invoice or receipt
- ▶ Description of defect

We reserve the right to refuse guarantees if this information has been removed or changed after purchase of the product or is not provided with the claim.

2. Repairs by unauthorised specialist companies

Guarantee repairs may only be carried out by specialist companies authorised by invicon. There is no claim for reimbursement of costs or elimination of defects for repairs carried out by other third parties,

3. Remaining time

In the case of repair or replacement, the guarantee claim for the repaired or replaced product is only valid for the remaining time of the original guarantee period.

7. GUARANTEE

4. Use in other countries

If the device is intended to be used in a country other than that for which it was originally manufactured, and changes therefore have to be made to the product for it to meet the technical and/or safety-related standards, these adaptations are expressly excluded from this guarantee.

5. No guarantees

The following are excluded from the guarantee:

- ▶ Maintenance and inspection services and repair or replacement of parts in accordance with the list of wear parts;
- ▶ Transport and travel costs, and costs arising from installing and removing the device;
- ▶ Damage caused by misuse of the product or use for another purpose;
- ▶ Damage caused by incorrect installation of the product or insufficient maintenance;
- ▶ Damage caused by lightning, water, fire, force majeure, war, incorrect supply voltage, insufficient ventilation, lack of decalcification, insufficient care and cleaning or non-compliance with the prescribed rules for use and maintenance services;
- ▶ Damage caused by use of the product outside of the given specifications;
- ▶ Damage provably caused by consumable or wear material that is not original;
- ▶ All damage that did not occur directly on the device and other consequential damage.

The guarantee expires if:

- ▶ changes have been made to the product by the operator or a third party;
- ▶ the product is still operated despite the presence of a functional or usage defect.

6. Individual claims

This guarantee is product-related and can be claimed in the guarantee period by any person who has acquired the device legally.

7. Guarantee of spare parts

Spare parts are guaranteed against material, processing and manufacturing defects for a period of 6 months in accordance with the provisions of these guarantee conditions.

7. GUARANTEE

8. Legal claims for defects

The purchaser's legal claims for defects are unaffected by this guarantee.

9. Final provisions

Should individual provisions of these guarantee conditions become ineffective either in full or in part, this shall not affect the validity of the remaining provisions. The fully or partially ineffective regulation shall be replaced by a regulation that comes as close as possible to the commercial success of the ineffective regulation.

This guarantee is exclusively subject to Austrian law and excludes the UN Convention on Contracts for the International Sale of Goods. The place of jurisdiction is Feldkirch, Austria.

8. DISPOSAL

In accordance with the applicable legal framework, our devices are disposed of as per the B2B specifications for proper disposal of commercial end products. Please contact your specialist retailer for further information.

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Representative authorised to compile the relevant technical documentation:

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